HUMAN CARE

Your Life. Your Way.

Reconditioning manual/
Rekonditioneringsanvisning

neXus

Reconditioning manual

neXus 3

To ensure that your walking aid is functioning properly and safely, an annual inspection should be done only by an authorized agent. The following maintenance should be done regularly:

Cleaning

When cleaning, use water and mild detergent.

When machine washing, remove the two soft seat cushions by loosening the red Nexus handle and unbuttoning the cushions from the seat.

After washing, the cushions are mounted on the seat again.

Machine wash 60 ° C for 7-8 minutes can be used, observe the finishing requirements. Avoid high pressure cleaning on bearings (wheels, forks).

NOTE: Do not use naphtha, gasoline or other solvents.

Post treatment

Ensure that all moving parts, such as bearings, are well lubricated. Ensure that your walking aid is dry to avoid rusting.

Chassis

Check weldings and screws. If any corrosion is evident, have your rollator inspected by an authorized agent. Replacement of parts or product may be required.

Handle

Regular assessment by an authorized agent is required for any necessary adjustments. The handle should stay in the "locked" position when pushed down and easily move up when "squeezed" towards the grip. Change the grip as needed.

Brakes

Control that both brakes and parking brakes are working.

Wing nut (Star handle)

Control function. Ensure it is securely fastened.

CE-marking

Make sure that the CE-marking is still there and readable.

Wheels

Make sure the wheels are not worn-out and are spinning easily without any noise.

Front forks

Make sure that the ball bearings are firmly attached.

Folding mechanism

Control function.

Seat

Make sure the seat remains in place and the screws secure and that the seat pad is properly mounted.

Stability

Ensure that all wheels are level and steadily on the ground and that there is no chassis distortion.

Test driving

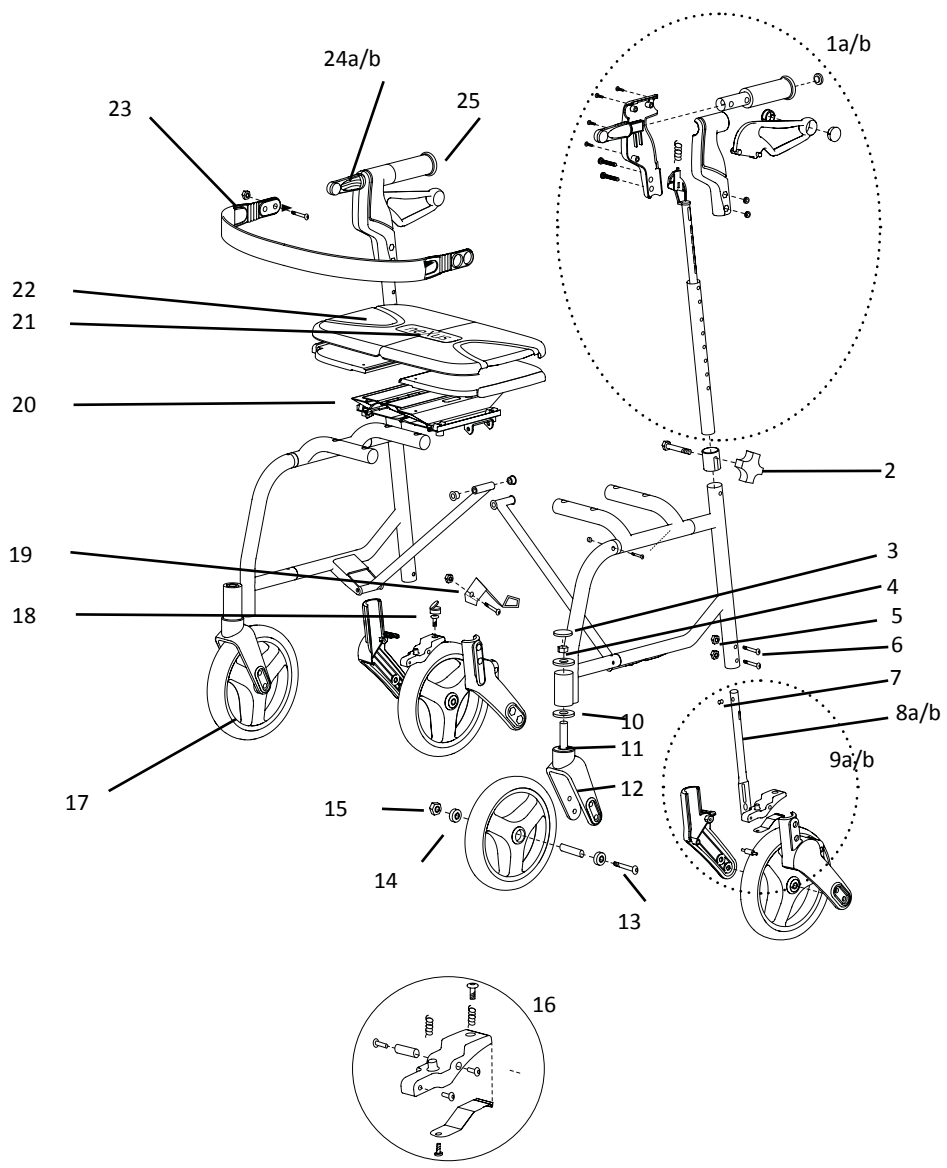
Do a final control check by test driving.

Rollator neXus - Reconditioning manual

Spare part list

No	Description	Qty	Art. no	Other information
1a	Brake handle Right- compl.		P48535 P48555	Regular Tall
1b	Brake handle Left- compl.		P48545 P48565	Regular Tall
2	Star Handle and bolt kit		89096	For models after 2009
3	Bolt Cap		4823	
4	Front Fork Stem Nut		H48025	
5	Rear Fork Tubing Nut		H48031	
6	Rear Fork Tubing Screw		H48030	
7	Lock Button		4866	
8a	Lower brake rod Left- assembly		76013 76016 76018	super low low standard
8b	Lower brake rod Right-assembly		76015 76017 76019	super low low standard
9a	Rear fork Left- assembly		AS48013 AS48015 AS48017	super low low standard
9b	Rear fork Right- assembly		AS48014 AS48016 AS48018	super low low standard
10	Front Fork Bearing		H48026	2 pcs is needed
11	Front Fork Stem Bolt		H48024	
12	Front Fork		4825	
13	Axle Bolt		H48020	
14	Wheel bearing	2	H48022	
15	Axle Nut		H48021	
16	Brake Pad Kit (complete)		76011	
17	Wheel complete		P48012	Incl. bearings and bearing spacer
18	Plastic screw cap cover		H45003	
19	Curb climber		4879	
20	Folding seat mechanism		P45007	
21	Seat Handle red		P45008	
22	Seat Pad	2	4540	2 pcs (set)
23	Backstrap kit		P48010	
24a	Backstrap clamp, left	1	4880	
24b	Backstrap clamp, right	1	4881	
25	Rubber handle		4870	
26	Wire Basket		4510	
27	Soft bag	5	80501Q5	Only sold in 5 pack
28	Assembly kit soft bag		90276	Incl. metal brackets and clamps

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