HUMAN CARE

Your Life. Your Way.

Accessory manual

Carl-Oskar, Rebel, Carl-Oskar RA, Staffan RA, Rebel RA,
Carl-Oskar TW/gåbord, Staffan TW/gåbord, Rebel TW/
gåbord, Svea TW/gåbord.

Content

Accessory overview	3	Tillbehörsöversikt	13
Assembly Tray	5	Montering bricka	15
Assembly Slow Down Brake	5	Montering släpbroms	15
Assembly cane holder	6	Montering käpphållare	16
Assembly One Hand Brake	7	Montering enhandsbroms	17
Assembly Back Support	7	Montering ryggband	18
Assembly Curb Climber	8	Montering steghjälp	19
Assembly Splash Guard	9	Montering stänkskydd	20
Assembly Oxygen Holder	10	Montering Syrgashållare	20
Assembly Basket	11	Montering korg	22
Attaching small and big bag	12	Attaching small and big bag	23

Accessory Manual (ENG)

Carl-Oskar, Rebel, Carl-Oskar RA, Staffan RA, Rebel RA, Carl-Oskar TW, Staffan TW, Rebel TW, Svea TW.

Accessory Overview

Image	Description	Part no	Image	Description	Part no
	<p>Tray</p> <p>Practical for resting light weight items.</p>	<p>C-O & Rebel: 57 cm: 10651, 62cm, 67cm and 72cm: 10602</p> <p>C-O RA/TW, Rebel RA/TW: 10602</p> <p>Staffan RA/TW, Svea TW: Not available</p>		<p>Backrest</p> <p>Gives support to the back while seated.</p>	<p>C-O and Rebel: 57 cm, 62cm and 67 cm models: 10621. Part no. for 72cm model: 10655</p> <p>All other models: Not available</p>
	<p>Slow-down brake</p> <p>Prevents the rollator from moving too fast and enables a controlled walking speed for the user.</p>	<p>C-O & Rebel: 10620</p> <p>C-O RA/TW, Rebel RA/TW: 10620</p> <p>Staffan RA/TW, Svea TW: Not available</p>		<p>Curb climber</p> <p>Helps raise the rollator onto a curb.</p>	<p>C-O & Rebel, C-O RA/TW, Rebel RA/TW, Svea TW: 10612</p> <p>Staffan RA/TW: Not available</p>
	<p>Cane holder</p> <p>Simplifies the transportation of a cane or umbrella.</p>	<p>C-O & Rebel, C-O RA/TW, Rebel RA/TW: 10603</p> <p>Staffan RA/TW: 40603</p> <p>Svea TW: Not available</p>		<p>Splash guard</p> <p>Protects legs and clothes. Sold in pairs.</p>	<p>C-O, C-O RA/TW: 10608</p> <p>All other models: Not available</p>
	<p>One-hand brake</p> <p>Allows the user to control brake functions with one hand.</p>	<p>C-O & Rebel, C-O RA/TW, Rebel RA/TW: 10652</p> <p>C-O RA/TW, Rebel RA/TW, Svea TW: 10653</p> <p>Staffan RA/TW: 10653 (low) 10654 (high)</p>		<p>Oxygen holder</p> <p>Holds oxygen tube firmly in place.</p>	<p>C-O & Rebel, C-O RA/TW, Rebel RA/TW: 10615</p> <p>Staffan RA/TW: 40605</p> <p>Svea TW: Not available</p>
				<p>Basket with handle</p> <p>Basket ideal for shopping.</p>	<p>C-O & Rebel, C-O RA/TW, Rebel RA/TW: 10601</p> <p>Staffan RA/TW: 40601</p> <p>Svea TW: Not available</p>


Accessory Overview

Image	Description	Part no
	<p>Bag Kit of two bags that offers storage space for shopping or personal belongings.</p>	<p>C-O & Rebel: Carl-Oskar and Rebel 57 and 62) Part no: Carl-Oskar and Rebel 67 and 72) Part no:</p> <p>C-O RA/TW, Rebel RA/TW: Not available</p> <p>Staffan RA/TW, Svea TW: Not available</p>

Carl-Oskar, Rebel, C-O RA/TW, Rebel RA/TW

The tray rests on the seat of the rollator (see picture). Wrap tray clip on one side and push down firmly into position with both hands. Remove from the seat by pulling up from the back side of the tray.

For the RA/TW models the tray is only suitable for models with a seat. Maximum weight: 5 kg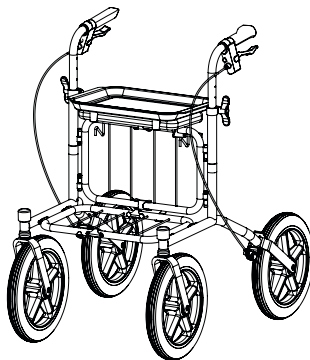

Assembly Slow-Down Brake


Assembly (Picture 1)

Wrap the two parts of the slow-down brake around and above the rear fork. Assemble into position with the screws and the locking nuts taking care not to strip the screws. Ensure the slow-down brake is centered over the wheel.


Adjust the level of the brake wheel up or down so that the wear washer on the wheel brake can move without obstruction. Securely fasten into position.

Adjustment (Picture 2)

Height adjustment of the brake wheel is done by turning the adjustment knob located at the top of the brake clockwise or counter-clockwise. Turning clockwise, the pressure is harder on the wheel resulting in a more inert movement. During use, the wheel should move in a consistent pattern without intermittent obstruction (wheel should rotate at a slower pace evenly). Lock into desired position by tightening the nut located at the base of the knob.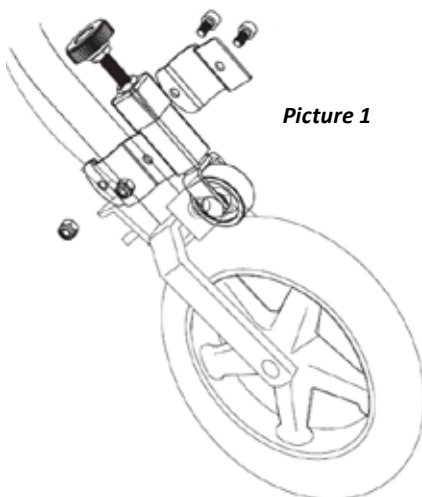

Picture 1

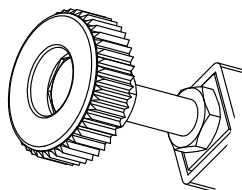
Picture 2


Assembly - Upper Part

Verify that you have all components before assembly. The cane holder comes in two parts - an upper and a lower part - both of which are composed of two separate pieces.


1. Open the clamp that is secured to the frame tubing so that the clips come closer to each other.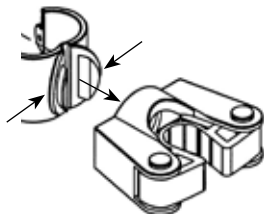

2. Fit the cane holder hole through the clips of the attachment. Release slightly so that the cane holder is secure and place the clamp around the handle frame.

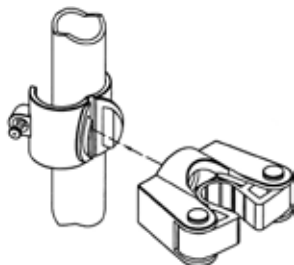
3. Close the clamp and tighten with the screw, taking care not to strip the screw. Note! Make sure that the attachment is secured.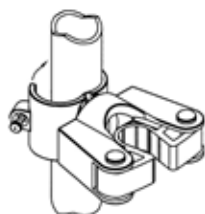


Assembly - Lower Part

1. Open the clamp that is secured to the frame tubing so that the clips come closer to each other.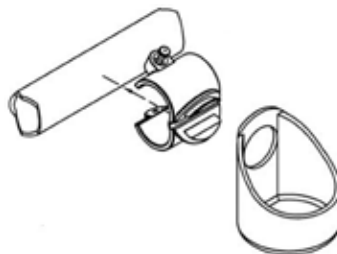

2. Fit the clips into the cup hole and release slightly to secure. into position. Turn the cup into the correct position to hold the cane/umbrella bottom tip before securing onto lower frame.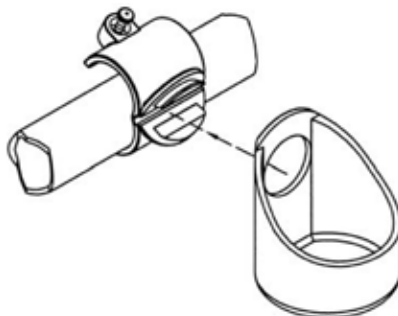

3. Close the clamp and tighten with the screw, taking care not to strip the screw. Note! Make sure that the cup is secured.


Assembly One-Hand Brake

ENG

The one-hand brake allows the user to brake the walking-aid with one hand.

The single hand brake can be assembled afterwards or be ordered as pre-assembled unit. When ordering pre-assembled, please state on which side of the walking aid the handle should be placed.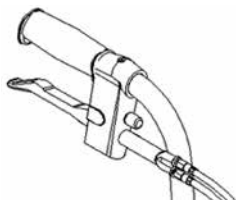

Disassembly


1. Disassemble the cables from the wheels on both sides of the rollator. Disassemble the end cover (1) from the brake housing (5) and remove the cables and cable housings from the rollator.

2. Remove the brake lever (2) on the side that will not be used as a one-hand brake.

Assembly

1. Attach the two adapters (6) into the end cover (7).

2. Pull through the two assembled cables and reattach into the end cover (7). Pull the cables through the housing (5) and around the washer (3) and secure it to the brake lever with the brake handle pin (4). Attach the brake lever to the housing.
3. Assemble the shorter cable assembly to the bottom wheel on the side where the one-hand brake is placed. The longer cable assembly should be attached to the other wheel.
4. Adjust the brake with the adjustment screw for increased / decreased braking action.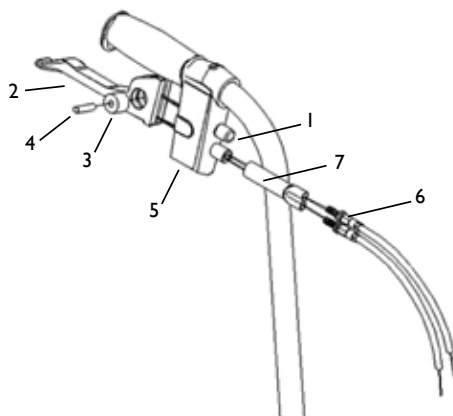

Assembly Back Support

ENG

The back support comes with the following parts:


A: Holder (2/ea)

B: Holder (2/ea)


C: Screw M4x14 (4/ea)

D: Screw M4x20 (4/ea)

E: Nut M4 (8/ea)

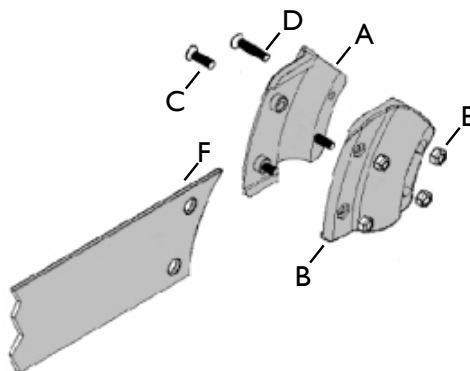
The back strap can be cut to achieve different lengths.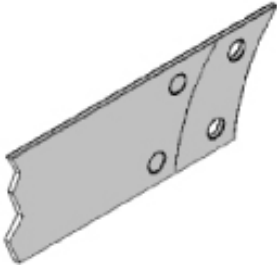

Fold one end over onto the other side to the desired length (so they overlap each other). Using the existing holes from the top belt as a guide, mark the hole positions and the cut required for the end. Note! The distance from the hole to the end should be 7mm.

Use a hole puncher to punch the holes in the positions marked. Cut the end using a belt cutter.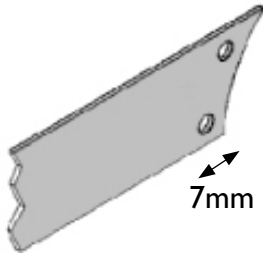


Assembly onto a Rollator

Line up clamps on both sides of the rollator tubing at the bend as shown. Place the backstrap between the two clamps, lining up the holes.

While holding the backstrap, press clamp A with clamp B and secure in position with screws. Take care not to strip the screws while tightening. Screw D is used to secure closest to the frame and screw C is used to secure the backstrap.


Do not rest anything on the back support as damage may occur. Check frequently for any loose, cracked or corroded plastic. Check frequently to ensure screws are tightened firmly.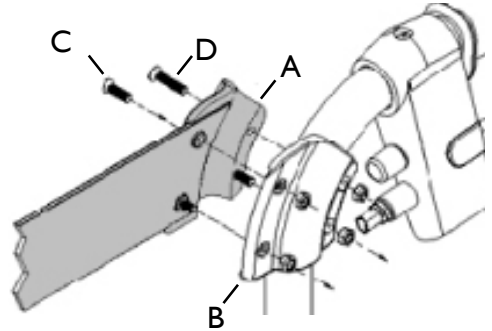


Assembly Curb Climber

ENG

1. Remove existing bolt and nut.
2. Assemble the curb climber on the inside of the rollator (between the wheels) on either the right or left side according to the suggested heights indicated in the table.
3. Secure in position with new bolt and nut.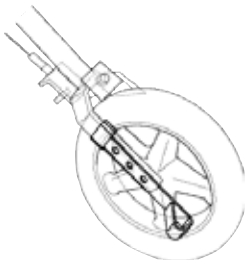


The curb climber has three holes for height adjustment which allows it to be suitable for the Carl-Oskar, Rebel and Svea, and also allows height adjustment from ground level to accommodate the user's requirements.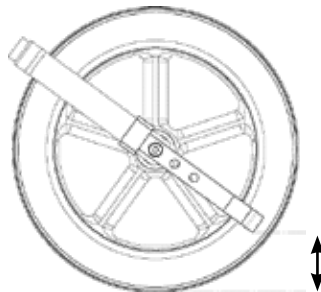

Height Adjustment Table (suggested distance between curb climber and ground).

Carl-Oskar	Rebel/Svea
61mm	-
76mm	22 mm
91mm	37 mm

Note! Bolt and nut must be replaced when adding a curb climber. Do not re-use existing bolt and nut.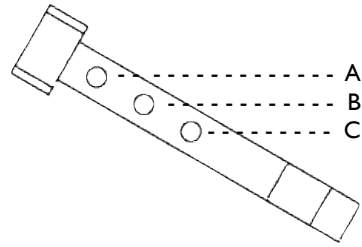


Using the Curb Climber

Place the foot onto the curb climber to act as a lever and raise the rollator slightly upwards towards you to raise the front wheels. Place onto a curb. Do not step on the curb climber!

Assembly Splash Guard

ENG

Place the splash guard onto the rear wheel and secure on the inside of the rollator by securing the attachment on to the rear fork.

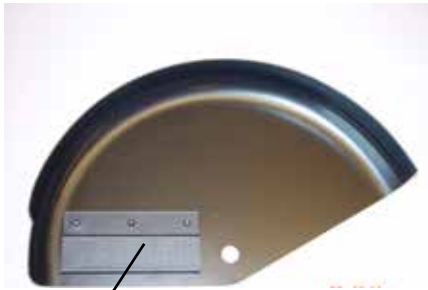
Attachment

Assembled on the rollator


Assembly - Oxygen Tank


The oxygen holder is delivered complete with two preassembled straps.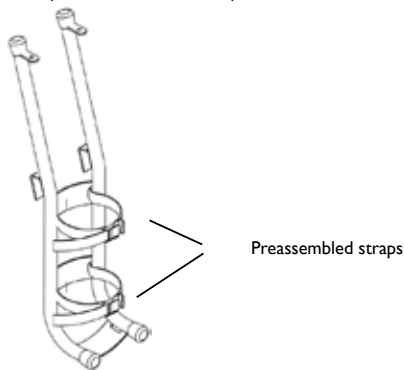

1. Loosen the straps before the tank is put in place by pushing the lock, on the strap, and pulling out the loop to preferred size.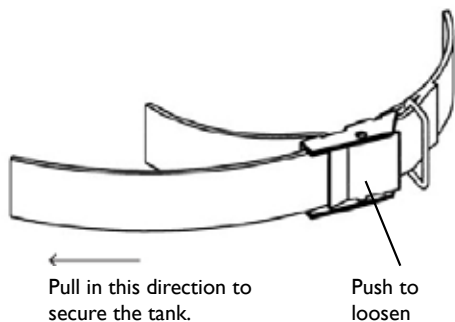



2. Tighten both straps when the tank is in place to secure it.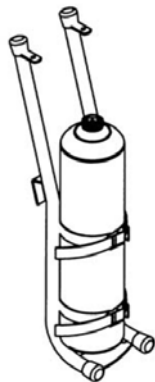


Assembly - On Rollator

1. Tilt the oxygen holder and pull it through the upper gate. The holder shall be placed in the middle of the gate.

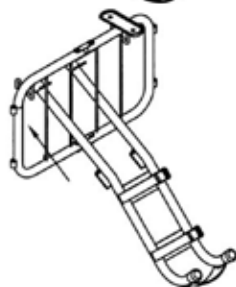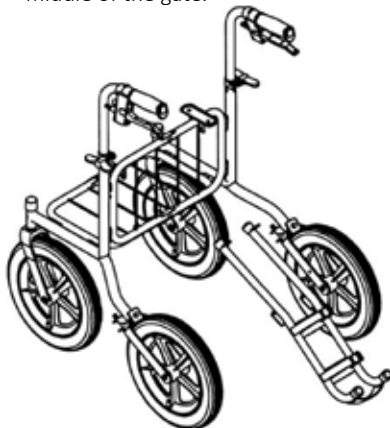
2. Hook the oxygen holder to the lower tubes of the gate and tilt it back (vertically) so that the upper attachment on the holder slides under the upper tubes of the gate.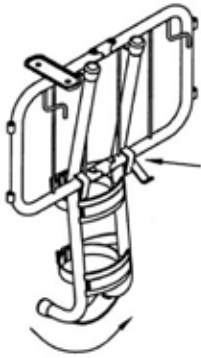

3. Ensure the oxygen holder is secure before use.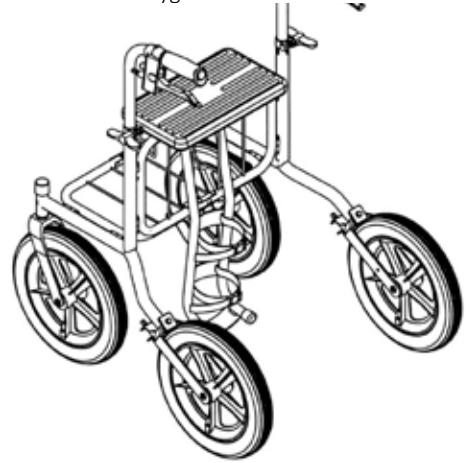


Assembly Basket

ENG

C-O & Rebel, C-O RA/TW, Rebel RA/TW:

Place and attach the basket onto the hooks with the straight side of the basket against the gate (see picture). Note! Basket can only be used when the rollator is in the unfolded position. Max load 10kg.

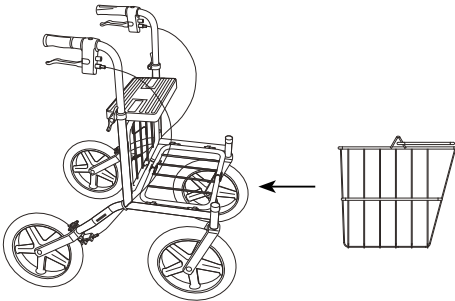Staffan RA/TW:

Assemble the basket onto the rollator by hanging the basket on the hooks on the front of the rollator.

Attaching the small bag

The small bag is placed with the opening towards the user for easy access to personal belongings.

Center the bag onto the middle of the rollator seat.

Using the velcro straps located on either side of the bag, secure tightly onto each frame handle.

Attaching the big bag

Place the bag into the hard basket, pushing down so it is a snug fit. Ensure that the back of the bag flap is secured as shown.

Ensure that the front is secured as shown.

Snap all the push buttons around the bag to secure into position.

Attach the basket with the inserted bag onto the rollator.

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