HUMAN CARE

Your Life. Your Way.

Diana II MLA 90555 / Diana II ELA 90556

User manual / Bruksanvisning / Gebrauchsanweisung /
Gebruikershandleiding

Content

ENGLISH

User manual	3
Intended use	4
Safety warnings	5
General maintenance and use	6
Safety instructions	7
Parts	10
Accessories	11
Technical specifications	12
Product label	14
Product symbols	14
Description of parts	15
Operating instructions	25
Maintenance instructions	28
Troubleshooting	31
Recycling	32

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User manual

Important!

Always read the user manual before using your product and keep this manual for future use.

Symbols

WARNING symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the product itself.

INFORMATION symbol provides recommendations and information for the correct and easy use.

SAFE WORKING LOAD symbol indicates the maximum user weight for this particular lift. Do not exceed this weight under any circumstances.

CE MARKING symbol indicates that the product is a medical device according to the definition of a medical device in EU Directive 93/42/EEC (MDD) and/or Regulation (EU) 2017/745 (MDR).

MEDICAL DEVICE symbol indicates that the product complies with the requirements of EU Directive 93/42/EEC (MDD) and/or Regulation (EU) 2017/745 (MDR).

RECYCLE according to local regulations.

INSTRUCTIONS do not use this product until you have read and understood the instructions.

Product modification is not permitted. Always check before use that the product is not damaged and shows no signs of wear and tear.

At Human Care, we continue to develop and improve our products in order to offer our customers the highest quality. We reserve the

right to make design changes without prior notice.

Always make sure that you have the latest version of the user manual. It is available on our website, www.humancaregroup.com.

Please contact your local dealer if you have any questions about the lift and its use.

Intended use

The Diana II is an extremely stable, electrically operated lift for lifting clients. It has a compact, foldable chassis so that it can even be used in spaces with limited room to maneuver. Our extensive range of slings provides the required support.

Due to its compact size, the Diana II is ideal for use in small spaces, both at home - narrow doorways and toilets/bathrooms - as well as in healthcare institutions where other necessary medical devices can often clutter an area.

The Diana II has a large lifting range. This means that the lift has a large reach: it is able to lift far. This is particularly important for positioning a person comfortably at the back of a chair or wheelchair. The lift has a large lifting range (lifting high/low), which makes the Diana II a versatile lift that can be used in many situations.

The Diana II can be used with both a 2-point hanger bar and a 4-point hanger bar. The 2-point hanger bar is ideal for use with strap loop slings. The 4-point spreader bar has hooks for attaching our E-clip slings. The universal E-clip is a proven system for fastening a sling to a lift, thereby significantly reducing the risk of error. The clip is placed around the snap hooks of the hanger bar and closes with an audible 'click'. It is a signal to the caregiver that the clip is securely attached and cannot unexpectedly become loose. Using the 4-point hanger bar

allows the caregiver to place the client into the desired position, varying from recumbent, semi-recumbent and sitting.

This electrically operated lift provides support to clients while they are lifted from their bed, in/out of a chair or wheelchair, on/off the toilet and even from the floor. The leg spreading allows the lift to be placed around a chair/toilet to get as close as possible to a client for a more comfortable transfer. You can choose between operating the legs using foot pedals (Manual Leg Adjustment = MLA) or an electronic system (Electric Leg Adjustment = ELA).

When the lift is not in use, it can be folded easily so it can be stored or taken to another location.

Do not use the lift if:

The safe working load is exceeded.

The clips of the sling are not securely and properly attached.

The product is damaged.

You have not read and understood all the safety warnings and instructions.


Safety warnings

Only trained professionals can make the decision whether or not this lift is appropriate to use. This product can only be used under the supervision of a caregiver. Never leave the client unattended while in the lift.

If you do not understand any of the warnings, safety precautions or instructions, please contact a care provider, distributor or technical staff member (if applicable) before attempting to use this product. Using this product without having the proper knowledge may result in injury or damage.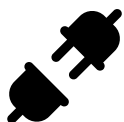

Disconnect the product and remove the battery during maintenance work.

Product modifications are not permitted. Maintenance and servicing may only be carried out by qualified personnel.

Use only the parts that are supplied with the product. Only use spare parts supplied by Human Care.

Product modifications are not permitted. Maintenance and servicing may only be carried out by qualified personnel.

Do not touch parts of the products that may have an electrical current (leakage) at the same time as the patient.

Maximum safe working load
160 kg/352 lbs.

General maintenance and use

Check the Diana II before each use to make sure it is not damaged in any way. Do not use the Diana II if there are signs of wear and tear or damage on any part of the lift.

This lift may only be used by qualified personnel.

The Diana II may only be used on an even, clean and hard-floored surfaces that are free of obstacles. The lift should never be used on a sloping floor or ramp.

In certain circumstances, more than one caregiver may be necessary to assist a particular client.

The client's abilities should be assessed before using this product. Only use this product for its intended use.

The Diana II is designed to support only one client at a time. Never use the lift to lift a client and an item of furniture at the same time.

Only use other Human Care products with the Diana II.

This lift is for indoor use only. It is not suitable for use in areas with a high humidity (e.g. shower rooms, hydrotherapy), extreme temperature or high pressure environments.

A specialist must approve use of the lift for a specific client and this must be recorded in writing.

Before using the lift for a client, it is strongly recommended to practice using the lift with colleague-caregivers, until the caregiver feels comfortable using the product.

The correct type, size, material and design of accessories must be carefully selected by an expert, taking into account the safety and needs of the client.

Always charge batteries in a well-ventilated room. Check the charger regularly.

Safety instructions

- **Please read the instruction manual for both the lift and the lifting accessories before use. Lifting and transporting a person always involves a certain risk.**
- **It is important to fully understand the content of the manuals and to use the lift as per the instructions in the manuals.**
- **The lift may only be used by trained personnel.**
- **In case of doubt or if you have any questions, please contact Human Care or your dealer.**

Warning!

Certain conditions and surroundings may limit the use of the lift, e.g. thresholds, uneven surfaces, obstacles and deep-pile carpet. These conditions and surroundings may prevent the castors of the lift from working properly, may cause the lift to become unstable and/or may make it harder for the caregiver to use the lift. If you are not sure whether your care environment meets the requirements for proper use of the lift, please contact Human Care for help and advice.

CAUTION!

This user manual contains important information for the user of this product. All users of the lift must read and understand the content of the user manual and work accordingly. Always keep the user manual in a place where it is available to those persons using the product.

General safety precautions

Always attach the sling according to the instructions for the sling that has been supplied.

Always check that the safe working load of the lift is suitable for the client's weight.

Always carry out lifting activities according to the instructions in the user manual.

Never disconnect a control or safety device and never try to by-pass such a function because you think it might make it easier to operate the lift.

Never attempt to move the lift by pushing against the mast, boom or client. Always use the appropriate push handle to move the lift.

When transporting a client, never move faster than a slow walking pace (3 kilometers per hour - 0.8 meters per second).

Do not push the lift over an uneven or rough surface, especially when transporting a client.

Do not try to push/pull a loaded lift over an obstacle on the floor, which the castors cannot clear.

Never force a control/safety device. All controls are easy to use and do not require the use of physical force.

Never leave a loaded lift on a sloping surface.

Never use an electric lift in the shower.

Never take the lift down a flight of stairs, either with or without a client.

Before moving the lift, always check that it is disconnected from the charger, as this could damage the electrical connection.

Always make sure you follow these instructions before using the lift:

Always carry out a daily check before using the lift;

Always read the instruction manuals for the lift and the lifting accessories. Determine what you need to do before you start using the lift;

The caregiver should always familiarize himself/herself with the controls and safety devices before attempting to lift a client;

Only use slings that are recommended for use with this lift.

Ensure that the correct lifting accessories are being used in terms of type, size, material and design to meet the client's needs;

Make sure the lift accessories are not damaged;

Ensure that the brakes are intact; any missing or damaged brakes must always be replaced;

The lifting accessories are correctly attached to the lift;

Ensure that the clips of the sling are correctly clipped to the hanger bar after the sling has been fully extended, but before the client has been actually lifted.

Ensure the batteries are fully charged;

Parts

- 1. Hand control**
Controls the functions of the lift.
- 2. Boom**
Raises/lowers the client in a sitting position, always in combination with a sling.
- 3. Lift actuator**
Electric lift actuator for raising and lowering the boom.
- 4. Push handle**
Handle for the caregiver to move the lift.
- 5. Control box with emergency stop and battery**
Power pack.
- 6. Hanger bar (accessory)**
- 7. Foot pedal for manual leg operation**
- 8. Chassis**
Chassis with 4 castors.
- 9. Castors, rear**
Double castors with brake.
- 10. Castors, front**
Double castors without brake.
- 11. Electric leg operation**
Provides the option to position the legs around a chair or wheelchair, so that the lift can be positioned closer to the client in order to achieve a more comfortable lifting process.

Accessories


2-point hanger bar for loop slings:

Productnaam	Artikelnummer
HANGER BAR 2P35 (FOR MOBILE LIFT)	90567
HANGER BAR 2P45 (FOR MOBILE LIFT)	90557
HANGER BAR 2P60 (FOR MOBILE LIFT)	90568

Examples of Human Care loop slings:

Productnaam	Artikelnummer
Full Body Sling Net	25005-(size)
Toileting Sling	25085-(size)
Toileting Sling High	25090-(size)
Bath Sling	25035-(size)
Sit sling	25030-(size)

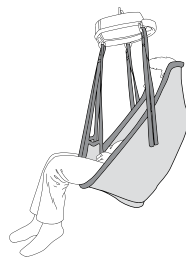
4-point hanger bar for slings with E-clip attachment:

Productnaam	Artikelnummer
HANGER BAR 4MSB	90558

E-Clip slings for 4-point hanger bar


Productnaam	Artikelnummer
Full Body Sling E-clip Net	25005C-(size)
Full Body Sling E-clip Soft	25010C-(size)
Columbus Sling E-clip Soft	25040C-(size)
Toileting Sling E-clip	25085C-(size)
Toileting Sling High E-clip	25090C-(size)

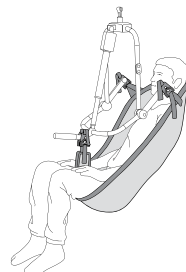All slings are available in different sizes to achieve a best possible fit with the client.

Technical specifications

Description	
Article number	90555 (MLA), 90556 (ELA)
Length of the frame	1280 mm / 50.4"
Width of the frame	620 mm / 24.4"
Height	1360 mm / 53.5"
Height of the push handle	966 mm / 38"
Lifting height (min.)	720 mm / 28.4"
Lifting height (max.)	1940 mm / 76.4"
Minimum width of legs	600 mm / 23.6"
Maximum width of legs	1110 mm / 43.7"
Height of legs (from the floor)	95 mm / 3.7"
Turning radius (min.)	1390 mm / 54.7"
Turning radius (max.)	1520 mm / 59.8"
Lifting speed (up)	34 sec.
Lifting speed (down)	33 sec.
Diameter of castors with brake	100 mm / 3.9"
Diameter of castors without brake	75 mm / 3"
Operation	continuous, short-term load (10%) max. 2 min. in operation (on) min. 18 minute break (off)
Control box	input voltage 100-240V ~, frequency 50/60 Hz, output voltage 24V, power consumption max. 400 mA
Battery capacity	2.9 Ah
Electric shock protection class	II, adapted part type type B
IP rating control box	IPX4
IP rating class battery	IPX5
IP rating main lift actuator	IPX4
IP rating hand control	IPX6
Maximum working load	160 kg / 352lbs
Product weight	45 kg / 99lbs
Noise level	52 dB
Frame material	powder coated steel

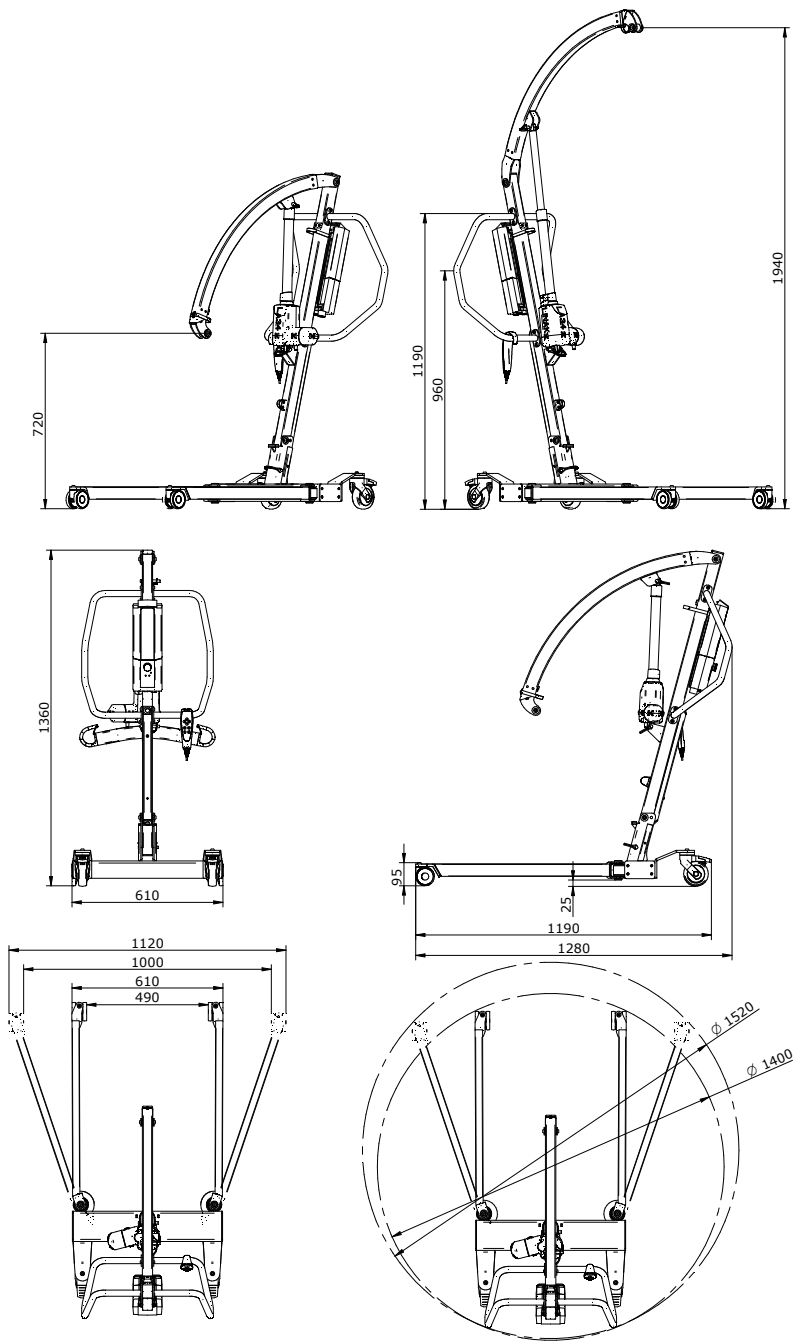Product label


The product label contains information about the lift and is located on the right-hand side, close to the mast.

In compliance with / Standards / CE

Human Care is an ISO 13485:2016 Swedish certified medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product has a CE marking in accordance with the Medical Device Directive 93/42/EEC and/or Medical Device Regulation (EU 2017/745). The CE mark label is at the product.

This product has been tested by an accredited test institution and meets the following necessary medical requirements: EN 10535, EN 60601-1 (electrical safety) and EN 60601-1-2 (EMC). This product carries the CE marking.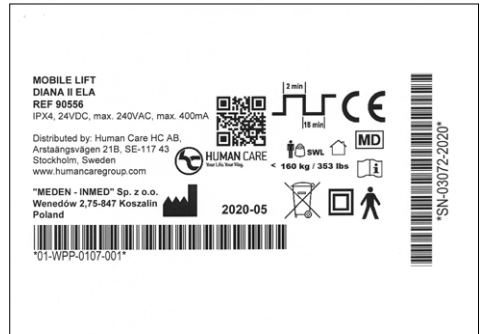

Product symbols

Symbols on the product and the product label:

	Product operating cycle
	CE marking
	MD Medical Device
	Product manufacturer
	Maximum working load

	For indoor use
	User manual: Please read the manual to familiarize yourself with its contents before using the product
	This product contains electrical and electronic components and must be recycled according to local guidelines.
	Electric shock protection class II
	Product type B: approved for physical contact with clients (IEC 60601-1)

Description of parts

Hand control

The Diana II is equipped with a hand control to control the various functions of the lift.

Depending on the version, two types of hand control are available.

1. Battery charge

The battery charge indicator has three status indications. The battery charge indicator is yellow or green until switched off (2 minutes after use).

For a full description, see page 18.

2. Service indicator

The service indicator (yellow indicator) flashes when it is time to carry out a service. The default setting is every 12 months / 8000 lifting cycles (as recommended by the EN10535 standard), whichever comes first. When it is time for a service, the service indicator stays on for 2 minutes after use. It is then switched off to save battery life. The system is still functional when the service indicator is on.

3. Overload indicator


When there is an overload, according to the predefined power interruption, the two LED lights will flash for 10 seconds.

4. Up


This button raises the lift arm, to help a client into a standing-up position.

5. Down

This button lowers the lift arm, to help a client into a seated position.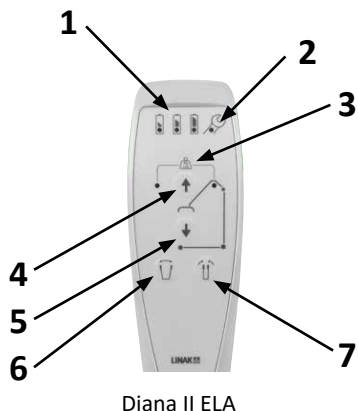

Diana II ELA

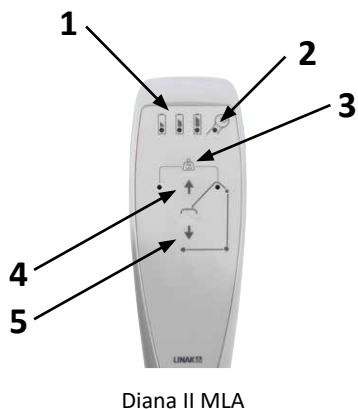Diana II MLA

6. Leg operation, open (Diana II ELA)

The legs can be spread so that the lift can be brought closer to a client sitting in a chair for a more natural lifting movement.

7. Leg spreading, close (Diana II ELA)

The legs can be closed to make it easier to maneuver and negotiate narrow passage-ways (e.g. doorways).

Emergency stop

In the event of an emergency, immediately press the emergency stop button.

Turn the button clockwise to unlock the emergency stop.

Electric emergency lowering function



If the button on the hand control for lowering the lift arm does not work, you can use the electric emergency lowering function. This can be activated by using a pointed object (e.g. a pen) to press the small button on the control box. You will hear an audible click. Keep pressing until the required height has been reached.

Mechanical emergency lowering function

If the emergency lowering function via the control box does not work, use the mechanical lowering function. This will safely lower the client in the event of a fault in the electrical system. Lowering can be done manually by using the lever located on the lower part of the main cylinder. To operate, grasp the red part and pull it up. Allow the boom to lower until the user can be safely removed from the lift.

Caution!

After using the mechanical or electrical emergency lowering function, you must immediately contact your supplier, dealer or municipality to schedule a visit by a service technician. The lift may only be used again after an authorized employee has carried out an inspection of the lift.

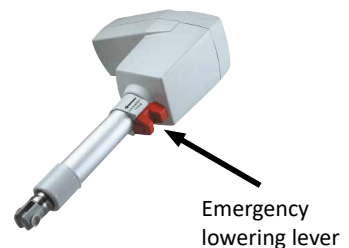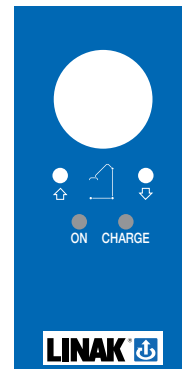Battery

Installing and removing the battery


Caution!

Do not use the lift for at least one hour before replacing the battery.

Never expose the battery to extreme temperatures. Never leave the battery charger in damp/humid areas.



Installing the battery:

- Align the bottom of the battery with the top of the controller.
- Push the top of the battery forward until you hear a click. The battery is now in place.
- Gently push the top of the battery forward to make sure it is securely attached to the holder.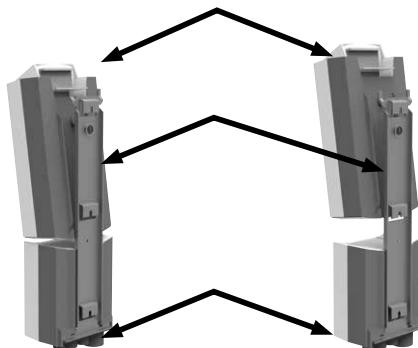

Removing the battery

- Press the emergency stop button.
- Hold the battery by the handle and press the release lever
- Remove the battery by lifting it off the holder

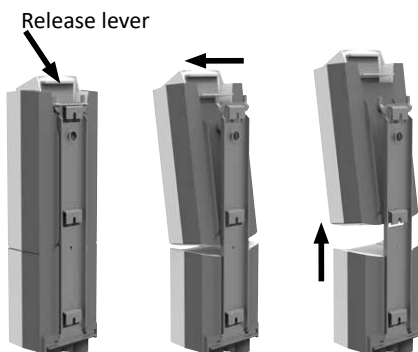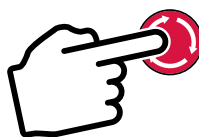Battery charge display and charging

Caution!

- The product cannot be used while charging.
- During charging, the lifting function is locked.
- Do not use the lift for 1 hour after charging. This will prolong the life of the battery.
- The battery needs to be charged continuously for at least 24 hour, in the following situations:
 - a) Before using the lift for the first time.
 - b) Before being placed in storage (up to 3 months) without a power supply connection.
 - c) Immediately after a storage period.

Battery charge indicator

The battery charge indicator has three status indications. The battery indicators are either yellow or green until switched off (2 minutes after use).

Battery status 1:

The battery is full – no recharging required (100-50%): the third indicator is green.

Battery status 2:

The battery is okay (50-25%) – recharging required: the second indicator is yellow.


Battery status 3:

The battery is low (less than 25%) – recharging required: the first indicator is yellow and a buzzer sounds whenever a button is pressed.





Battery status 4:

The battery requires recharging. Some functions of the lift are lost and it is only possible to lower the boom. When in continued use, the indicator flashes and an audio signal is emitted.

The battery charge is indicated on the hand control.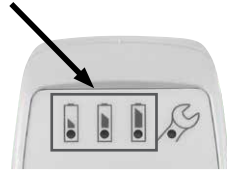


Battery level indicators on the hand control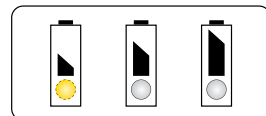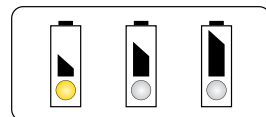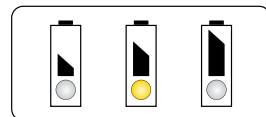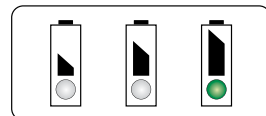

Charging the battery

If you need to charge the battery, you need to:

- Press the emergency stop button
- Connect the black charging cable that is supplied to the control box
- Then insert the plug into a wall socket that is located in a dry room.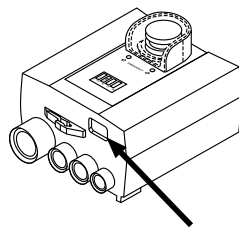

Connection for charging cable

When the rechargeable battery is connected, the LED display on the control box will come on.

The green LED indicates that the battery charger is connected to the mains. The yellow LED indicates the charging process. It takes approximately 5 hours to fully charge the battery.

Charging the battery with an external charger (optional).


You can always purchase an external charger, for example to have a spare battery to hand. To charge the battery, it needs to be inserted in the external charging station. Ensure that the battery is properly inserted into the charging station. A light will come on if it is attached correctly.

If the battery emits a short beep, it needs to be charged. If a green light comes on when the battery is inserted, the battery is full. If the orange light comes on, the battery needs to be charged. Charging will stop automatically when the battery is full. This means you cannot overcharge the battery. It is best to charge the battery every night.

The external charger can be easily mounted by means of the wall mounting plate. It can be mounted on the wall near a 230 V socket.


We always recommend you purchase a spare battery/charger, so that you will always have access to a full battery. If one battery is empty, you can continue to work using the second battery.

Location of LED on control box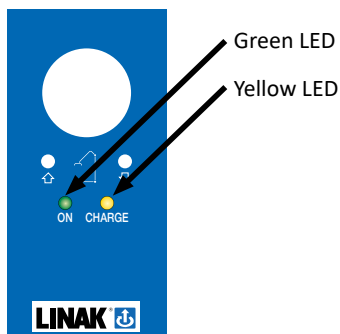

Green LED

Yellow LED

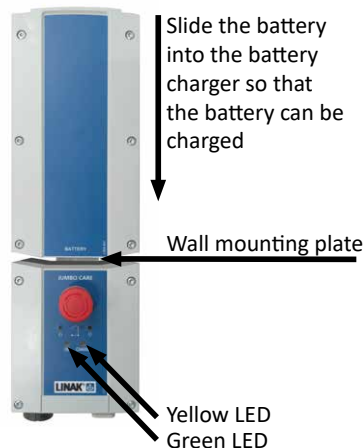Wall mounting plate

Yellow LED
Green LED

Charging the battery must not take place in a wet/humid environment (shower area).

Leg adjustment

The leg operation function is useful when you need to move the lift to a client on a chair or other piece of furniture. The width of the chassis can be adjusted, so that the legs of the lift can fit around furniture. This means that the lift arm and hanger bar are closer to the client, making it easier to attach the sling and therefore ensuring a more comfortable lifting process.

Caution!

Keep sufficient distance from people and objects when using the leg adjustment function.

The leg adjustment function of the DIANA II lift is available in two options. Mechanical leg adjustment (MLA) and

electric leg adjustment (ELA). For both these options the legs can be adjusted from 0° to 37°.

1) Mechanical leg adjustment


Use the appropriate foot lever to adjust the legs of the DIANA II MLA lift (fig. 16). Push down on the pedal on the righthand side to open the legs of the lift (fig. 16a). Push down on the pedal on the left hand side to close the legs of the lift (fig. 16b).

2) Electric leg adjustment (ELA)


Press the buttons on the hand control to open (1) or close (2) the legs of the lift (figure 17).

Using the brakes on the castors


The brakes (figure 1) are important components of the lift. The brakes are mounted on the rear two castors and prevent unwanted movement during use of the lift to ensure a stable starting position.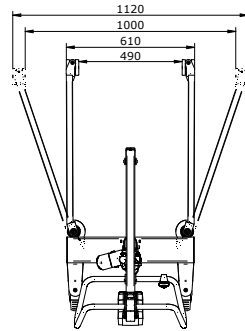

opens the legs

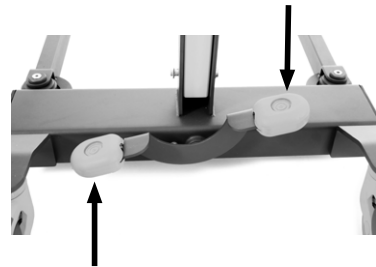closes the legs

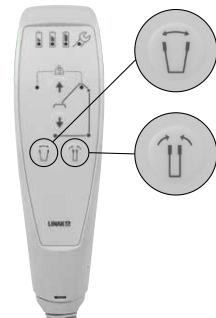The brake is applied by using your foot to push down the red lever.

The brake is released by using your foot to push up the red lever.

Folding/unfolding the lift




Locking pins

The DIANA II lift can be folded and unfolded quickly and safely using the universal locking pins, without the need for tools. It is important to unlock the pin during transport or maintenance of the lift by releasing the safety catch and then lifting it (figure 2a). To lock the pin, reverse the actions (figure 2b).

Assembly

After unpacking the Diana II, assemble the lift as follows:

1. Remove the locking pin (figure 3) while holding the upper part of the device;
2. Lift the upper part of the lift and lock it with the locking pin (figure 4);
3. Release the velcro strip from the lift arm (figure 5);

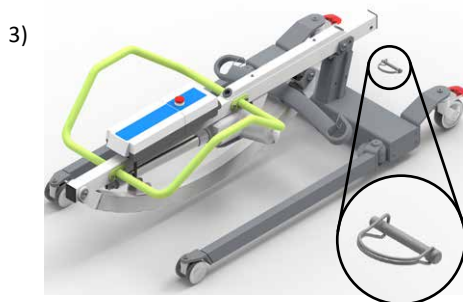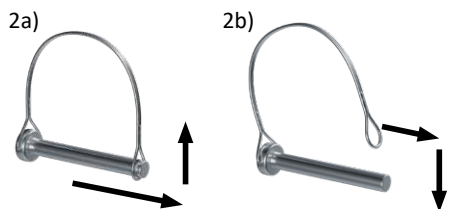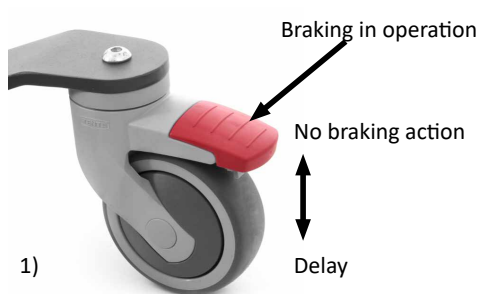Diana II

4. Lift the lift arm and release the velcro strip from the supplied actuator (figure 6);

5. Bring the lift actuator to where it needs to be secured and lock it with a locking pin (figure 7).

When this procedure has been completed, the DIANA II is ready for use.

Folding/unfolding the lift

The lift can be folded for easy storage or transport.

To fold the lift, follow the following steps:

1. Lower the lift arm to the lowest possible position and secure the 2-point hanger bar to the lift arm with the hook and loop strip. In case of a 4-point hanger bar: remove the hanger bar according to the instructions in the relevant manual;
2. Remove the top locking pin from the lift actuator (figure 8);
3. While holding up the lift arm, attach the lift actuator to the mast with a loose strap (figure 9);
4. Connect the rotating shaft of the boom to the mast using the locking pin (figure 10);

Caution!

Always lock the brakes when folding the device!

8)

9)

10)

Diana II

5. Remove the vertical locking pin at the bottom of the mast (figure 11);

11)

6. Tilt the top of the lift forward until the pin can be inserted and locked (figure 12);

Operating instructions

CAUTION!

The assembled boom may only be raised by the lift actuator. Manual lifting the boom could damage the lift actuator.

Do not use the lift in an environment where other radio frequency emitting devices are used. Like any other electronic device, the lift can use and emit radio frequency energy. If the lift is not used in accordance with the instructions in the user manual, it may cause interference with nearby devices. The manufacturer cannot guarantee that interference will not occur in a particular location. In order to check if the lift is the source of any interference, change its location or disconnect the battery. If you still notice or suspect a fault, please try the following:

- Move the lift to another location;
- Increase the distance between devices;
- Contact the service-desk.

Moving the Diana II

The push handle must be used for moving and maneuvering the Diana II.

Think about your posture:

- Walk behind the lift when pushing straight ahead.
- Start moving the lift by using your own body weight as much as possible.
- Keep your back as straight as possible.
- When you need to make a turn, keep one hand on the push handle and use your foot to gently push the leg of the base in the correct direction.




Caution!

Always fit the sling as indicated in the user manual for the sling.

Lifting the client


1. Position the lift with the correct hanger bar close to the client.
2. Check whether the client is in a safe position.
3. Lower the boom and hanger bar using the hand control to a position where it does not touch the client but it is possible to attach the loops or safety clips to the hanger bar without having to use physical force.
4. Attach the loops or safety clips of the sling to the hanger bar. Attach the loops or clips for the shoulders first, followed by the legs.
5. Carefully check each of the attachments. Using the hand control, raise the boom/hanger bar until there is some tension in the sling but do not start lifting the client yet. Check the attachments again.
6. Ensure the brakes are not engaged when lifting the client. The lift should be able to move to its centre of gravity.
Note: Brakes should be locked when lifting a client from the floor.
7. Lift the client by pressing the up button on the hand control. When using the 2-point strap hanger bar, the client remains in the same position. When using the 4-point hanger bar, the client can be moved into the preferred position: a recumbent, semi-recumbent and sitting position.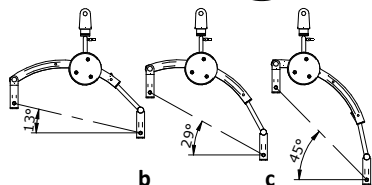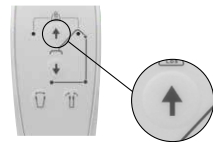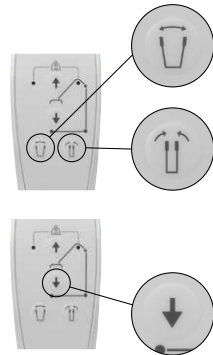

Transferring the client

1. Move the lift backwards so that the legs can be closed again (2nd button on the hand control, figure 1).
2. Start moving the client. For the comfort of the client it is better to keep to short distances.
3. Once you have reached your destination, you can open the legs of the lift (if necessary) before getting close to another object (e.g. chair/toilet).
4. Using the 4-point hanger bar, place the client in a sitting position when he/she is above the chair or wheelchair or into a recumbent position if the client is above his/her bed.
5. Lower the client by holding down the relevant down button until the client is in a safe position.

1)

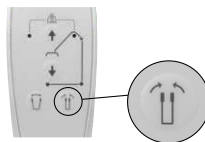Detaching the sling

1. Only detach the sling when the client is fully placed on a solid surface such as a bed or wheelchair.
2. Ensure that the hanger bar cannot hit the client and then detach the sling from the hanger bar.
3. Check that all connection points are loose before doing anything else.
4. Move the lift away and park it in a safe place.
5. When the lift is parked, the brakes of the castors can be applied.

2)

Maintenance instructions

Storage and transport

The lift must be kept in a cool and dry room. The climate conditions of the room must be within the following parameters:

- Temperature 10-40 °C (recommended 20 °C or lower);
- Humidity 30 ÷ 75%;
- Air pressure: 700 ÷ 1060 hPa.

Do not use the lift in environments where flammable materials may be present, e.g. in storage areas for flammable substances.

Cleaning

Cleaning is an important requirement to ensure that the lift works properly over a long period of time. Routine cleaning of the lift will suffice if it is used by the same client.

The lift will need to be disinfected if there are visible signs of material or possibly infected material (blood, stool, pus) or in situations requiring a high level of high infection control.

CAUTION!

The Diana II must be cleaned and disinfected every time a different client uses the lift before it can be used again.

CAUTION!

Before cleaning the lift, check that:

- All plugs are correctly connected.
- The battery is mounted on top of the control box.
- None of the electrical components show external damage. Failure to do this may result in water or cleaning agents to get into the lift and cause interference with the unit or damage to electrical components.
- Electrical components cannot be washed with water jets or high-pressure cleaners, etc. They can only be cleaned with a damp cloth.
- Stop using the lift if you suspect that water or liquid has entered the electrical components and report the incident immediately to the service center.
- Failure to comply with the above rules may result in serious damage to the equipment and could have other unforeseen consequences.

Cleaning contact surfaces

1. The surface of the handles must be cleaned and maintained as follows:
 - a) Undo the straps and remove all parts that do not belong to the lift.
 - b) Clean surfaces with mild and environmentally-friendly cleaning agents
 - c) The same applies to cleaning the hand control.
2. Accessible areas of the push handles and the frame can be disinfected using mild and environmentally-friendly detergents
3. The castors only need to be disinfected if there are visible traces of infected or potentially contaminated material.

Do not use:

- Pastes, waxes, sprays,
- Strong detergents, solvents and detergents containing solvents, alcohol and leather cleaners.

Using such agents can also cause hardness and cracks in the material.


Any changes to the surface sheen are not covered under the warranty!

Servicing

An annual service and maintenance by authorized personnel is required every 12 months.


Human Care cannot be held responsible for damage caused by willful damage, overdue maintenance and incorrect use, or any resulting consequential damage.

We recommend checking the lift regularly for loose parts. Regularly clean the castors.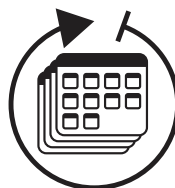

Maintenance

During long periods of inactivity, we recommend checking all electrical and mechanical parts once a month by carrying out a lift test without the client. Additionally, the charging cables should be visually checked for possible damage after each mechanical lift or after moving the lift to a different location.

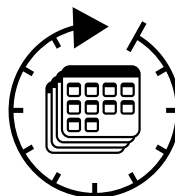Maintenance of the frame

The metal sections of the frame can be cleaned with a soft, damp cloth. Cleaned surfaces should be wiped dry each time. Do not use cleaning products containing alcohol.

All moving joints must be lubricated once every six months or when they start making a loud noise when used. The moving joints include:

- castors and actuators;
- bearing bushes and joints of the legs and boom.

As a lubricant, we recommend using commercially available penetrating and lubricating products (e.g. Wurth HHS 2000). Any spillage of such products must be removed immediately with a dry cloth.

Threaded connections should be inspected periodically – once every six months – and, if necessary, any tolerance must be remedied. Any inadvertent tolerances must be reported to the manufacturer's service center. The lift can no longer be used until the cause of this tolerances has been removed. Any tolerance in joints that cannot be remedied must be reported to the manufacturer's service department. The product can no longer be used until the cause of this slack has been removed.

Periodic electrical safety tests

A service is required every 12 months or 8000 lifting cycles, whichever comes first and every time after a fault/repair of the lift. Services, including the electrical safety test, must be carried out by a certified person.

Services should include as a minimum:

Check whether there is mechanical damage to the wiring;

Check whether there is mechanical damage to the housing of the controller, battery and actuator;

Check whether the height adjustment and leg adjustment buttons are working properly.

Troubleshooting

Problem	Solution
The lift does not respond to the hand control function	Check whether the emergency stop has been activated
	Check whether the battery is sufficiently charged
	Check whether the hand control cable is connected
	Check whether the battery is correctly fitted
	Check whether the charging cable is connected
	Check whether the other wires are connected
	Check whether the service LED is flashing
	Check whether the lift responds to the raising/lowering button on the controller
	Contact the service department
The lift does not respond to the control box buttons	Check whether the emergency stop has been activated
	Check whether the battery is sufficiently charged
	Check whether the battery is correctly fitted
	Check whether the charging cable is connected
	Check whether the other wires are connected
	Check whether the service LED is flashing
	Contact the service department
The lift is not charging	Check whether the emergency stop has been activated
	Check whether the battery is sufficiently charged
	Check whether the battery is correctly fitted
	Check whether the charging cable is connected
	Check whether the service LED is flashing
	Contact the service department
The lift stops while lifting the client	Check whether the overload indicator is flashing
	Check whether the battery is sufficiently charged
	Check whether the service LED is flashing
	Check whether the lift responds to the raising/lowering button on the controller
	Manually lower the lift
	Contact the service department
The lift is making strange noises (cracking, crossover, etc.).	Contact the service department
The lift cannot be moved	Check whether the brakes on the rear castors are locked
	Contact the service department

If the error messages persist, stop using the lift immediately and contact the dealer for further instructions.

Recycling

Diana II is an electronic appliance covered by Directive 2002/96/EC on waste electrical and electronic equipment (WEEE). The product label shows the WEEE symbol, which consists of a crossed-out wheeled bin. This label indicates that the batteries must be collected separately at the end of the lift's lifecycle. This requirement applies to the European Union. Do not dispose of these products in household waste.

Caution!

Batteries and electronics contain lead and must be recycled according to WEEE 2002/96/EC.

Metals

Electrical and electronic waste (WEEE)

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