HUMAN CARE
Your Life. Your Way.

Accessory manual

Parade

Content

ENGLISH

Accessory overview	2
Assembly Tray	3
Assembly cane holder	3
Assembly Back Support	4

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Accessory overview

Tray

Practical for resting light weight items.
Part no. 10664

Cane holder

Simplifies the transportation of a cane.
Part no: 10675

Back support

Supports the back when resting.
Part no. 10621

Curb climber

Helps raise the rollator onto a curb.
Part no: 10661


Basket

Basket ideal for shopping.
Part no: 10662

Assembly Tray

ENG

Assemble the tray on to the seat of the rollator as according to picture. Remove the tray from the seat by pulling in the back of the tray.

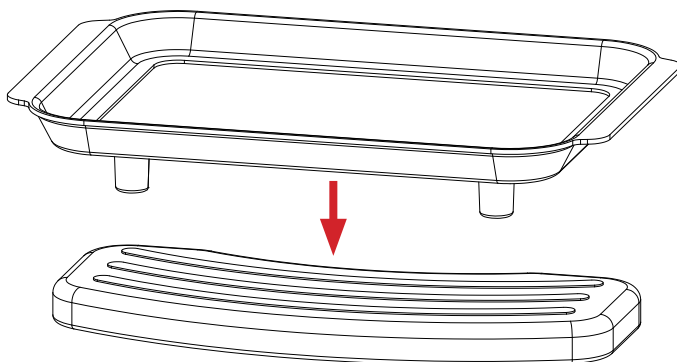Assembly cane holder

ENG

The cane holder is delivered in two main parts, an upper and a lower. These are in turn composed of two parts, verify that you have received them.

1. Determine which side of the rollator the cane holder should be placed. Both the upper (bracket) and lower (cup) part is mounted in the same way.

Figure 1

2. Place the parts around the pipe and tighten the screws (Figure 2).

NOTE! Make sure the cup/bracket is fixed.

Figur 2

Assembly Back Support

ENG

The back support comes with the following parts:


A: Holder (2st)

B: Holder (2st)


C: Screw M4x14 (4st)

D: Screw M4x20 (8st)

E: Nut M4 (8st)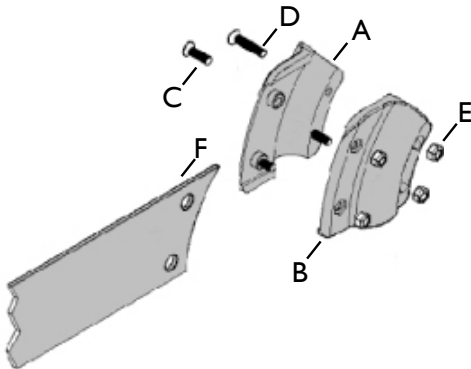


The back strap can be adjusted to different lengths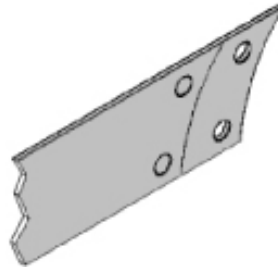

Fold both ends of the back strap (so they overlap each other) until preferred length is achieved.

Draw the radius and the holes on the underlying part and cut it along the marking.

Control measure the new end. The distance from the hole to the radius shall be 7mm.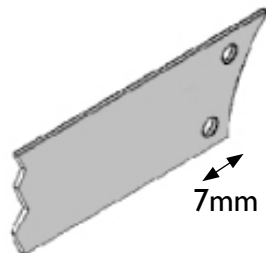


Assembling of the back support to the rollator

Place the holders (A and B) to the pipe, where the bending begins, and put the end of the back strap between them.

Push the strap against the holder A so the two protruding cylinders push through the straps hole. Press the two holders against each other and tighten with the screws. The screw C is for the front holes where the back strap is fixed. The screw D is for the rear holes.

Pay attention to cracks and loose parts. Make sure the back support is firmly tightened by pulling it.

The rollator must NOT be hanged, carried or lifted in its back support.

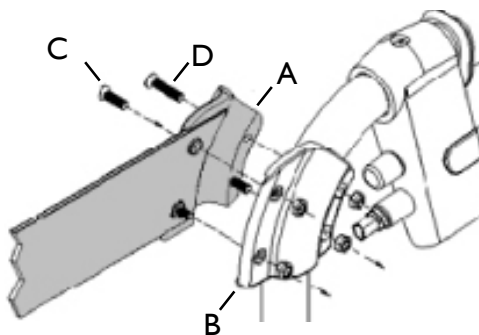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