Gleittuch Slide

Gebrauchsanleitung · Manual
Bruksanvisning · Käsimkirja · Gebruiksaanwijzing
Mode d'emploi

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1 Allgemeine Informationen

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Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of slides.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

Slides are made from a particularly low-friction and hard-wearing polyamide, allowing for easy movement. The Slides are produced as tubes.

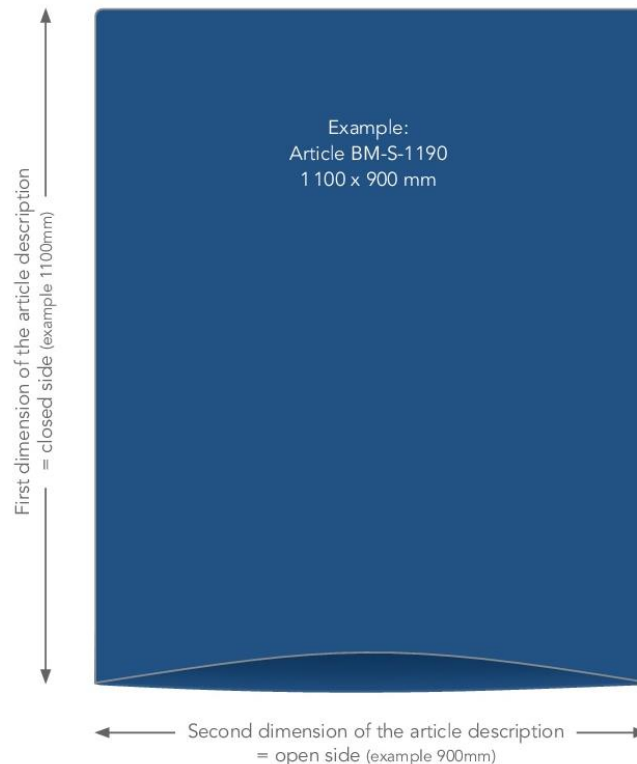
The Slides are part of a product line which is used specifically to aid in the transfer of patients in home care and other health care facilities, as well as in hospitals.

The product may only be used for the specified purpose and under the specified conditions.

The Slides are a safe transfer aid, which allows for a transfer which is pleasant and comfortable for the patient as well as being simple and back-friendly for the users.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Item No.: BM-0560 Slide
Item No.: BM-0652 Slide
Item No.: BM-0752 Slide
Item No.: BM-S-0790 Slide
Item No.: BM-S-0907 Slide
Item No.: BM-S-0910 Slide
Item No.: BM-S-0911 Slide
Item No.: BM-S-1170 Slide
Item No.: BM-S-1190 Slide
Item No.: BM-S-1370 Slide
Item No.: BM-S-1390 Slide
Item No.: BM-S-1970 Slide

Special sizes are possible on request!

1.2 Accessories

No accessories are available for this device.

1.3 Scope of delivery

Slide
User manual

1.4 Storage, transport

Store/transport at -5 C – 40 C
Other: protect from sunlight

1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **three years** with average wash cycles. However, the effective service life may vary depending on the frequency and intensity of use and the number of washing and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- The smooth surfaces significantly reduce resistance due to friction during repositioning. Check the transfer surfaces carefully to ensure they are fixed in place securely (e.g. wheelchair, bed).
- Ensure that the transfers are performed by the users in a back-friendly manner.
- Weight limits must be observed (150 kg).
- After the transfer / repositioning, remove the sliding mat from under the patient. If applicable, ask about products which can also be used for "bedding" applications.
- Transfer products are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.
- The use of non-standard cleaning agents or bleaching agents, alcohols, fabric softeners or other chemical laundry additives may lead to permanent deterioration of the sliding properties.

2.1 Symbols used

| | | | |
|---|----------------------------|---|-------------------------------|
|  | 60°C laundry |  | max. load capacity see item 6 |
|  | Do not bleach |  | Manufacturer |
|  | Dry at low temperatures |  | Batch no. / lot no. |
|  | Do not iron |  | Identification code |
|  | Do not dry clean |  | Conformity mark |
|  | Warning |  | Item number |
|  | Observe the user manual |  | Medical device label |
|  | Do not use fabric softener | | |

2.2 Function check

Transfer products are also subject to a natural ageing process. Inspect the equipment regularly and carefully, in particular before each use.

Visual inspection

Ensure that material, seams, handle loops and buckles are undamaged and in functional order.

Sliding properties

Place the product on a firm surface
Pull your hands over the product with pressure
Good glide - easy gliding
Low glide: no gliding/moving possible

➤ **Defective products may no longer be used!**

3 Using the product

3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since Slides are suitable for all applications involving people who require assistance. Thanks to the reduced friction, the capabilities of the patient can be put to use when Slides are used. The effort required on the part of the user is reduced.

This product may be reused.

Before reusing the product, be sure to perform a thorough function check (see item 2.2) as well as cleaning or disinfection of the product. (see item 5)

3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Observe the safety information under item 2.

If the muscle tone is elevated due to transfer movements being performed too quickly in patients who are anxious, are pain-sensitive, have a perceptual disability or are disoriented, then alternatives have to be developed. Please contact our experts if this is the case.

3.3 Application risks

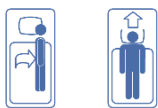
- Movements are performed too quickly → risk of falling
- Jerky movements → risk of elevated tone

3.4 Areas of application

The product can be used in home care and other health care facilities as well as hospitals.

On account of their water-repellent material properties, the Slides BM-0560, -0652 and -0752 are suitable for use in wet areas or in combination with the sliding board.

The soft versions of the Slides are suitable for in-bed transfers. (Item no. BM-S-0790 and up)

3.5 Application illustration

3.6 Application

Slides make it a lot easier to move patients, since they reduce friction to a minimum.

Slides are available in various sizes and versions, the correct size is determined on the basis of the patient's functional capacities and the desired change in position. They are tubular Slides in which the specified length is the respective length of the closed side. Thanks to their sliding properties, they can be used in any direction.

The Slides not only allow the patient to be moved within the bed, they can also be used for many position changes and transfers while recumbent or sitting. They support the movement transitions and are not intended to remain underneath the patient. The Slide can be combined effectively with the Sliding Board, the Turning Disc, a Raising Aid or the Support Belt. After use, the slides can be hung up by the label.

3.6.1 Placing the Slide, option A

- The Slide is pushed crosswise under the patient's pelvis.

3.6.2 Placing the Slide, option B

- Place the patient on their side.
- Fold 1/3 of the Slide underneath on the open side.
- Place the Slide against the patient's back, the position may vary correspondingly.
- Place the patient on their back.
- Pull the folded under third through sideways underneath the patient.
- The patient should be positioned in the centre of the Slide.

3.6.3 Moving toward the head

Prerequisites: The legs can be flexed. Use of an Anti-Slip Mat is recommended in this case!

- Flex the patient's legs.
- The patient can move up by extending the legs, on their own or with the user's assistance.

If the legs cannot be flexed, then an additional sliding mat is required (e.g. PM-2010).

3.6.4 Moving to the side

Prerequisites: none

The flexibility of the Slide allows for sideways pulling, pushing and lateral positioning.

3.6.5 Removal

After positioning has been completed, the sheet can be removed without moving the patient.

- Support the patient with one hand
- The sheet can be pulled out from underneath the patient with the other hand.

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

3.7 Approved accessories, combination with other products

Slides are used in combination with the following products:

- PM-7010/7011 Non-Slip Mats
- PM-8010 Bed Wedges
- PM-5020/30/50 Sliding Board

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

3.7.1 Sets

| Item No. | Ingredients set |
|----------|-----------------------------|
| PM-4225 | PM-4222 Raising Aid Pegasus |
| | PM-6080 Raising Aid Quick |
| | BM-0652 Slide |
| PM-5031 | PM-5030 Sliding Board |
| | BM-0652 Slide |

Please observe the user manuals of the other items as well.

4 Product marking / CE

The Slide is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

5 Cleaning, care and disinfection

| Cleaning | Desinfection (if necessary) |
|---|---|
| 60° washing machine | Recommended disinfectants for wipe disinfection: <ul style="list-style-type: none"> - Trionic® (DGHM/VAH approval, further information on request) - Mikrobac® Tissues* - Mikrobac® forte* - Bacillo® 30 Foam - Bacillo® 30 Tissues - Bacillo® 30 Sensitive Tissues - Mikrobac® Virucidal Tissues* |
| Mild detergent | |
| Do not use cleaning agents with bleaching ingredients or chemical laundry additives! Otherwise the product properties may deteriorate permanently. | |
| Wash with similar colours only | * wipe with a cloth soaked in water after disinfection to avoid residues or layer build-up. Please always follow the manufacturer's manuals for use when wiping disinfection. |
| Recommended detergent for industrial washing machines: <ul style="list-style-type: none"> - - Maximo Liquid as single detergent | |
| Low dryer temperatures (45° - 50° C) | |

Be sure to follow the care instructions on the label of each product as well as the manuals for use.

6 Technical Data

| Item no. | Material | Colour | Load-bearing capacity |
|----------|-----------------------|--------|-----------------------|
| BM-0560 | Polyamide, latex free | blue | 150 kg |
| BM-0652 | Polyamide, latex free | blue | 150 kg |

| | | | |
|-----------|-----------------------|------|--------|
| BM-0752 | Polyamide, latex free | blue | 150 kg |
| BM-S-0790 | Polyamide, latex free | blue | 150 kg |
| BM-S-0907 | Polyamide, latex free | blue | 150 kg |
| BM-S-0910 | Polyamide, latex free | blue | 150 kg |
| BM-S-0911 | Polyamide, latex free | blue | 150 kg |
| BM-S-1170 | Polyamide, latex free | blue | 150 kg |
| BM-S-1190 | Polyamide, latex free | blue | 150 kg |
| BM-S-1370 | Polyamide, latex free | blue | 150 kg |
| BM-S-1390 | Polyamide, latex free | blue | 150 kg |
| BM-S-1970 | Polyamide, latex free | blue | 150 kg |

| Item no. | Description | Dimensions in mm | Weight | UDI |
|-----------|-------------|------------------|--------|-------------------------|
| BM-0560 | Slide | 520 x 600 x 0,5 | 55 g | (01)04250355306526(10)x |
| BM-0652 | Slide | 600 x 520 x 0,5 | 55 g | (01)04250355306524(10)x |
| BM-0752 | Slide | 700 x 520 x 0,5 | 60 g | (01)04250355307521(10)x |
| BM-S-0790 | Slide | 700 x 900 x 0,5 | 90 g | (01)04250355307903(10)x |
| BM-S-0907 | Slide | 900 x 700 x 0,5 | 90 g | (01)04250355309075(10)x |
| BM-S-0910 | Slide | 900 x 1000 x 0,5 | 130 g | (01)04250355309105(10)x |
| BM-S-0911 | Slide | 900 x 1100 x 0,5 | 140 g | (01)04250355309112(10)x |
| BM-S-1170 | Slide | 1100 x 700 x 0,5 | 125 g | (01)04250355311702(10)x |
| BM-S-1190 | Slide | 1100 x 900 x 0,5 | 140 g | (01)04250355311900(10)x |
| BM-S-1370 | Slide | 1300 x 700 x 0,5 | 150 g | (01)04250355313706(10)x |
| BM-S-1390 | Slide | 1300 x 900 x 0,5 | 190 g | (01)04250355313904(10)x |
| BM-S-1970 | Slide | 1950 x 700 x 0,5 | 220 g | (01)04250355319708(10)x |

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 2 years. **It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.**
- The company Petermann is not liable for accidental or consequential injury or material damage.

8 Disposal information

There are no specific disposal regulations for Slides, they can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de

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