


Foldable  
4/6 grips  
Flexible



A transfer from bed to shower stretcher as well as lifting a patient from the floor is possible with the lifting mat. In addition, a patient lying in bed can be moved to the head end and to the side. The sliding surface significantly decreases the frictional resistance between the client and the transfer surface. The lateral sections can easily be folded inwards together with the grips, and with the client positioned on his side, the transfer sheet can easily be placed underneath the client. The inner section of the transfer sheet can be removed so that it can be washed or disinfected.

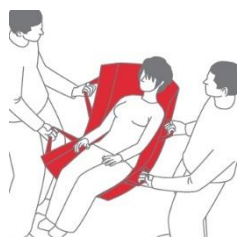

## Technical specification

Product Name	Material	Colour	Max Lifting weight	Guarantee	Storage/transport
Lifting Mat	Polyamide, latex free	red	200 kg	3 years	5 °C – 40 °C protect against sunlight

Item no.	Dimensions in cm	Weight	UDI
PM-4012	110 x 54	1240 g	(01)04250355340122(10)LOT
PM-4016	150 x 53	2100 g	(01)04250355340160(10)LOT

### Hygiene and maintenance

The product doesn't require any special maintenance. It can be machine washed at 60°C and dried at low temperatures (45° – 50°C). We recommend washing with mild detergents. The use of cleaning agents with bleaching ingredients or other chemical laundry additives, also as fabric softeners may lead to permanent deterioration of the product properties.

**The inner reinforcing components must be removed!**

When necessary, we recommend wipe disinfection with the product Trionic® or comparable products.



Made in EU