

Compact  
Manual  
Light



The Alpha Stand-up Aid Pegasus is used to support a person after a fall to assist them from a supine position from the floor to a comfortable seated position. Depending on the clinical condition, this can be performed by themselves or with help of a care giver. It is very compact, safe and lightweight to handle. This raising aid is ready for immediate use at home or as an ideal aid for utilization to provide support and help. The seat of the Stand-up Aid Pegasus has a minimum receiving height of 8 cm. The raising aid reaches a seat height of 42 cm.



## Technical specification

Product Name	Material	Colour	Max Lifting weight	Guarantee	Storage/transport
Raising Aid Pegasus	Drive mechanism/Handle/ Frame: <ul style="list-style-type: none"> <li>➤ - high grade stainless steel and spring steel</li> </ul> Seat/Back: <ul style="list-style-type: none"> <li>➤ Polypropylene</li> </ul> Handle box: <ul style="list-style-type: none"> <li>➤ plastic white</li> </ul>	white	135 kg	3 years	5 °C – 40 °C protect against sunlight

Item no.	Dimensions in cm	Weight	UDI
PM-4222	Standing 36 x 64 included back 50	10,4 kg	(01)04250355342225(10)LOT

### Hygiene and maintenance

The Petermann Raising Aid Pegasus is made from rust-proof components and therefore doesn't require any special maintenance. A visual inspection for defects is required before each use. For sanitary reasons, please clean the device on a regular basis. You can use common liquid household cleaner. After cleaning, all moving parts such as the threaded rod, gears, etc. can be coated with an oil-free lubricant (Vaseline) using a brush. If you do not wish to perform the inspection, cleaning and disinfection of your Raising Aid Pegasus yourself, you can send your device to our shop.



Made in Germany