



The Support Belt is part of a product range with a wide range of possible applications. This product is used in particular when raising and/or transferring clients in home care and other health care facilities and hospitals. It is a safe transfer aid, which allows for a pleasant and comfortable transfer for the client and being back-friendly and straightforward for the caregiver. The interior of the Support Belt is lined with a non-slip material, so it clings to clothing, and vertical and horizontal grips make the handling easier. The robust outer material is made of

nylon and prolongs the service life of the belt. This belt has a safe working load (SWL) of up to 200kg/440lb.



## Technical specification

| Product Name        | Material              | Colour | Max Lifting weight | Guarantee | Storage/transport                        |
|---------------------|-----------------------|--------|--------------------|-----------|--|
| <b>Support Belt</b> | Polyamide, latex free | red    | 200 kg             | 3 years   | 5 °C – 40 °C<br>protect against sunlight |

| Item no. | Dimensions in cm | Weight | UDI                       |
|----------|------------------|--------|---------------------------|
| PM-6013  | 101 x 12         | 450 g  | (01)04250355360137(10)LOT |
| PM-6014  | 117 x 12         | 520 g  | (01)04250355360144(10)LOT |
| PM-6015  | 148 x 12         | 600 g  | (01)04250355360151(10)LOT |

### Hygiene and maintenance

The product doesn't require any special maintenance. It can be machine washed at 60°C and dried at low temperatures (45° – 50°C). We recommend washing with mild detergents. The use of cleaning agents with bleaching ingredients or other chemical laundry additives, also as fabric softeners may lead to permanent deterioration of the product properties.

When necessary, we recommend wipe disinfection with the product Trionic® or comparable products.

