



The Alpha Lifting Seat is ideal for stable seated positioning when transferring persons down staircases and other transfer situations where there is no room for mechanical aids. The weight of the client can be distributed over up to five persons, even of different heights, to save their backs. The stable safety handles make the client transfer safe and easy

Technical specification

Product Name	Material	Colour	Max Lifting weight	Guarantee	Storage/transport
Transfer Seat Swing	Polyamide, latex frei	red	200 kg	3 years	5 °C – 40 °C protect against sunlight

Item no.	Dimensions in cm	Weight	UDI
PM-4035	38 x 32 x 55	2300 g	(01)04250355340351(10)LOT

Hygiene and maintenance

The Transfer Seat Swing can be wet cleaned. Use warm water and a mild cleaning agent. If applicable, wipe the product off with a moist cloth and then let it dry thoroughly. Don't use any scouring or bleaching agents or any sharp-edged objects for cleaning. When necessary, we recommend wipe disinfection with the product Trionic® or comparable products. The use of bleaching agents or alcohol can permanently impair the properties of the Transfer Seat Swing.



Made in EU