



The Alpha Leg Belt supports the transfer of a seated patient into a wheelchair, onto the toilet, a chair or car seat, or for similar low level transfer situations. The leg belt works well in combination with sliding boards, turn discs, raising aids or the belts. It makes the transfer much more back-friendly for the caregiver, easier and less intrusive for the patient. It facilitates the range of movement and significantly increases the safety of the transfer. The leg belt is gently placed around the legs of the seated client, below knee level, and fastened with the Velcro fastener. The caregiver then gently places her/his own foot between the patients feet. During transfer, the Leg Supports locks into position according to the patients endurance and allows the care giver to stabilize the rotation by placing light counter-pressure on the Leg Support. The Leg Support can be extended by the Leg Support Extension.



Technical specification

Product Name	Material	Colour	Max Lifting weight	Guarantee	Storage/transport
Leg Belt and Extension	PES/Carbon, latex free	blue / red	200 kg	3 years	5 °C – 40 °C protect against sunlight

Item no.	Dimensions in cm	Weight	UDI
PM-3005	100 x 22	345 g	(01)04250355330055(10)LOT
PM-3007	50 x 22	155 g	(01)04250355330079(10)LOT

Hygiene and maintenance

The product doesn't require any special maintenance. It can be machine washed at 60°C and dried at low temperatures (45° – 50°C). We recommend washing with mild detergents. The use of cleaning agents with bleaching ingredients or other chemical laundry additives, also as fabric softeners may lead to permanent deterioration of the product properties.



Made in Germany