



The Alpha Non Slip Cushion is placed underneath the thighs of a client seated in bed. A gentle and comfortable supporting and securing function is achieved through the non-slip properties of the surface.

It significantly prevents the client from sliding down in the bed while sitting

Technical specification

Product Name	Material	Colour	Max Lifting weight	Guarantee	Storage/transport
Non Slip Cushion	Polyethylen, latex free	blue	200 kg	3 years	5 °C – 40 °C protect against sunlight

Item no.	Dimensions in cm	Weight	UDI
PM-8050	40 x 50	285 g	(01)04250355380500(10)LOT

Hygiene and maintenance

The Non Slip Cushion can be wet cleaned. Use warm water and a mild cleaning agent. If applicable, wipe the product off with a moist cloth and then let it dry thoroughly. Don't use any scouring or bleaching agents or any sharp-edged objects for cleaning. When necessary, we recommend wipe disinfection with the product Trionic® or comparable products. The use of bleaching agents or alcohol can permanently impair the properties of the product.

