



The Alpha Footstool provides a stable supporting surface for the feet, facilitates a higher knee position and reduces pressure on the thighs. It is used to bridge height differences. Smaller people in particular can require this support during transfers and toilet visit, so that safe mobilisation is possible for the client and the caregiver.

Adding the Footstool Cover makes the product easier to bring with you as well as store close by on a hanger. Moreover the cover makes the product more hygienic and easy to clean - simply wipe off the cover with cleaning solution and use again.



The Footstool has non-slip properties when under load and can easily be moved to the desired position when not under load. It can be applied on its own or in combination with the Alpha Turning Disc and our Sliding Boards.

Technical specification

Product Name	Material	Colour	Max Lifting weight	Guarantee	Storage/transport
Foot Stool	Polyethylen, latex free	blue	200 kg	3 years	5 °C – 40 °C protect against sunlight
Foot Stool Cover	Polyamide, latex free	grey		3 years	5 °C – 40 °C protect against sunlight

Item no.	Dimensions in cm	Weight	UDI
PM-8090	38 x 32	480 g	(01)04250355380906(10)LOT
PM-8091	38 x 32	110 g	(01)04250355380982(10)LOT

Hygiene and maintenance

The Foot Stool can be wet cleaned. Use warm water and a mild cleaning agent. If applicable, wipe the product off with a moist cloth and then let it dry thoroughly. Don't use any scouring or bleaching agents or any sharp-edged objects for cleaning. When necessary, we recommend wipe disinfection with the product Trionic® or comparable products. The use of bleaching agents or alcohol can permanently impair the properties of the product.

PM-8090 **CE** **MD** Made in Germany

The Foot Stool Cover doesn't require any special maintenance. It can be machine washed at 60°C and dried at low temperatures (45° – 50°C). We recommend washing with mild detergents. The use of cleaning agents with bleaching ingredients or other chemical laundry additives, also as fabric softeners may lead to permanent deterioration of the product properties.

When necessary, we recommend wipe disinfection with the product Trionic® or comparable products.

PM-8091 **CE** **MD** Made in Germany