

Easy
One-way-
glide
Comfortable



Sliding Mat Wheelchair / tubular sliding mat / PM-2070



Sliding Mat Wheelchair with fixation loops / open sliding mat / PM-2076



Sliding Mat Wheelchair with 4 hand grips / open sliding mat / PM-2078

The Alpha Sliding Mat Wheelchair is a safe transfer aid for clients who have slid forward in the (wheel)chair. The Sliding Mat Wheelchair can be used to return patients to a stable sitting position. Depending on the circumstances, the client can place themselves in a stable position independently or with assistance.

All Sliding Mats Wheelchair only slide in one direction and are placed between the client and the seat and remain in the chair during use. The sewn-on arrow indicates the direction of movement.



Technical specification

Product Name	Material	Colour	Max Lifting weight	Guarantee	Storage/transport
Sliding Mat Wheelchair	Cotton/natural thickening, latex free	anthracite/red	200 kg	3 years	5 °C – 40 °C protect against sunlight

Item no.	Dimensions in cm	Weight	UDI
PM-2070	50 x 45	440 g	(01)04250355320704(10)LOT
PM-2076	130 x 45	660 g	(01)04250355320766(10)LOT
PM-2078	135 x 43	580 g	(01)04250355320780(10)LOT

Hygiene and maintenance

The product doesn't require any special maintenance. It can be machine washed at 60°C and dried at low temperatures (45° – 50°C). We recommend washing with mild detergents. The use of cleaning agents with bleaching ingredients or other chemical laundry additives, also as fabric softeners may lead to permanent deterioration of the product properties.



Made in Germany